

User guide

Viking[®] Maxi extra wide over toilet frame

Code: 398



Intended use

Designed for heavy duty use, this maxi extra wide over toilet frame is made from steel, reinforced to take 300kg and pre-treated for rust protection. The over toilet frame is designed to assist a person on and off the toilet. There are customisation options available to meet the user's specific needs, please enquire with Cubro.

Safe use



WARNING! This user manual must be read before using the toilet frame. Severe injury or death may result if user instructions, maintenance instructions and product warnings are not followed.



WARNING! The decision to use this toilet frame should always be guided by a qualified Health Care Professional, who can also advise and assist with the correct height adjustment and usage instruction.


- Do not exceed the safe working load of 300kg.
- Ensure that the over toilet frame is stable on the floor and cannot slide.
- Inspect at regular intervals for wear and damage. Remove from use immediately if faults are found.
- Under no circumstances should the product be moved whilst occupied.
- Supervision or assistance when using the over toilet frame is recommended for users with limited mobility.
- The product is suitable for use in bathrooms or wet areas; it is not suitable for use in swimming pools.
- Use the over toilet frame only for the purpose it is intended for.

Product specifications

Safe working load	300kg		
Overall dimensions	W 780mm	D 610mm	H 750mm
Base dimensions	W 780mm		D 605mm
Seat dimensions	W 370mm		D 430mm
Seat height	550mm		
Armrest height	750mm		
Width between arms	650mm		
Top of seat to top of arms	210mm		
Product weight	8.5kg		

Instructions for use

How to use the over toilet frame

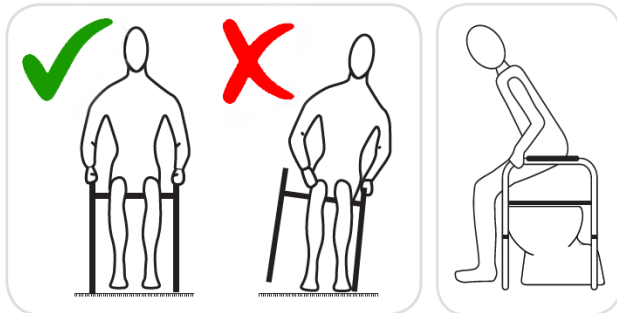

All four legs must be level on the floor and cannot slide.

To sit on the over toilet frame, move backwards until you can feel the seat behind your legs, place each hand on the armrest. Support your body and prevent the toilet frame from slipping away. Then lower yourself down.

When getting off the over toilet frame, move your bottom to the front edge of the seat ensuring your feet are firmly placed on the floor. Use the armrests to push yourself into a standing position leading with your head and shoulders.

Maintenance and cleaning

- Clean and disinfect using regular household cleaning products.
- DO NOT use abrasive powders or scouring pads to avoid causing corrosion.
- Should the equipment appear damaged, corroded or rusted remove from use immediately.

Servicing

For any servicing, maintenance and troubleshooting queries, please contact the Cubro Service Department:

0800 656 527 +64 7 578 7228

sales@cubro.co.nz

cubro.co.nz

149 Taurikura Dr, Tauriko, Tauranga, NZ



WARNING! Do not make any changes or modifications to this product without consultation from Cubro

Disposal

Products that can no longer be used are to be disposed of separately from household rubbish. This must be done according to the local and national regulations for environmental protection and raw material recycling. Please look at your local council's website for further information on the correct disposal.

Warranty

Cubro Ltd warrants the product to be free from defects in materials and workmanship from the date of purchase for a period of 1 year (12 months). Please refer to cubro.co.nz for terms and conditions.

For additional copies of this user guide or for a different format, please contact Cubro.

© Copyright 2025 Cubro® Ltd

ALL RIGHTS RESERVED: No part of this publication shall be adapted, modified, reproduced, copied, or transmitted in any form or by any means including written electronic, mechanical, reprographic, photocopying, or recording means. Furthermore, this publication shall not be stored in whole, part, adapted, or modified form, in or for any retrieval system of any nature without the written permission of the copyright holder. Applications for authorisation of a reserved right of copyright owner shall be made in writing to the publisher.

WARNING: The doing of any unauthorised act in relation to a copyright work may result in both a civil claim for damages and criminal prosecution.

31/7/25