


User guide

Viking® deluxe overbed table

Product code: 575

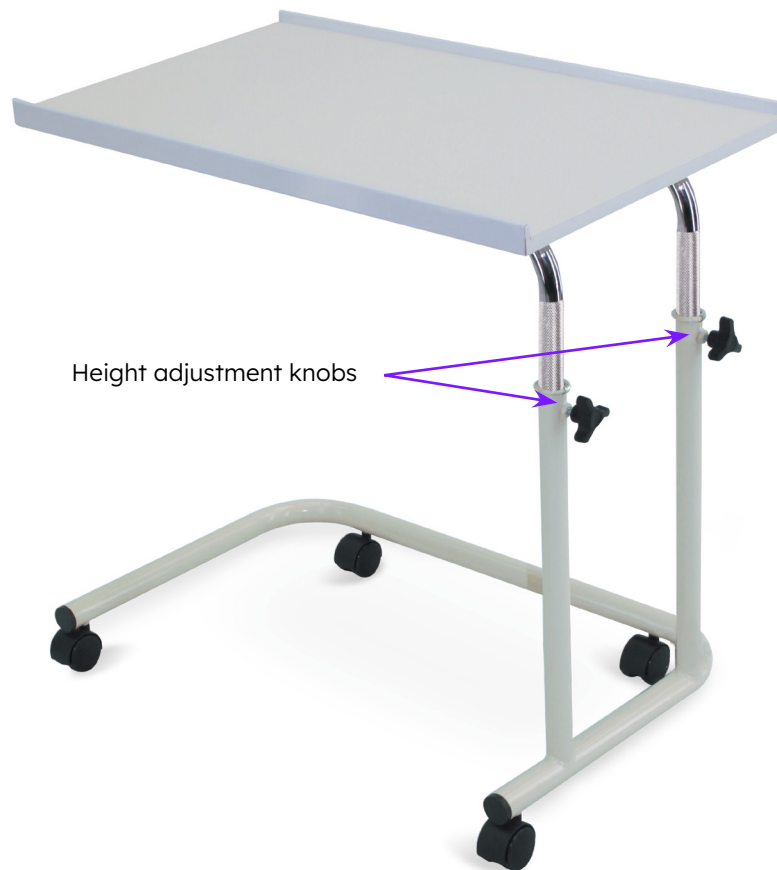

HEIGHT ADJUSTMENT:

To raise the tabletop:

- Loosen the locking knobs on the upright poles, lift tabletop to desired height. (At this stage the tabletop can be tilted at an angle).
- Retighten the locking knobs.

To lower the tabletop:

- Loosen the locking knobs on the upright poles.
- Lower tabletop to desired height and retighten the locking knobs.

CLEANING

- Clean using regular household cleaning products.
- Do not use abrasive powders or scouring pads.

