

Long Reach Comfort Wiper

The Long Reach Comfort Wiper is specially designed to clean the vaginal and rectal area after toilet use.

Instructions

- Pull around 24 " of toilet paper (around 6 sheets)
- Fold it in half
- Hold the Long Reach Tissue Wiper
- Tuck one end of the toilet paper into the oval recess (A)
- Wrap the toilet paper all around the top part of the wiper.
- Then tuck the end of the toilet paper on the same recess.
- The toilet paper will then be held in place for cleaning
- It is recommended to be used between the legs from front to back while sitting on the toilet.
- The soiled paper is ejected by pushing the button at the end of the Long Reach Tissue Wiper (B)
- Repeat the above instructions as needed.

Care:

- Do not immerse the Long Reach Tissue Wiper in water
- Clean with moist wipe or soft disposable cloth and mild soap.
- Pat dry with soft disposable cloth.
- For maximum hygiene, clean after each use.

FOR EXTERNAL USE ONLY

DO NOT USE IN SHOWER OR BATH

FOR HYGIENE PURPOSES, DO NOT SHARE BETWEEN PEOPLE

NOT TO BE USED ON ANIMALS OR YOUNG CHILDREN

