

Theorem Power Lift Recliners

Mobility Plus · 022 022 9956 · info@mobilityplus.co.nz · mobilityplus.co.nz

The five models stocked in New Zealand. All measurements in centimetres.

Model	Size	Max user weight (kg)	Seat width	Seat depth	Seat to floor	Chair width	Chair depth	Chair height
Mercer	Petite	136	45.7	45.7	45.7	81.2	96.5	101.6
Mercer	Standard	158.7	50.8	54.6	46.9	86.4	104.0	106.6
Studio	Petite	136	50.8	45.7	45.7	91.4	96.5	101.6
Studio	Standard	158.7	57.1	54.6	46.9	96.5	106.6	106.6
Seagrove	Petite	136	50.8	45.7	45.7	91.4	96.5	101.6
Seagrove	Standard	158.7	55.8	53.3	49.5	99.0	107.9	106.6
Aroki	Petite	158	45.7	45.7	45.7	85.1	97.8	104.1
Aroki	Standard	158	52.0	52.0	50.8	89.0	101.6	109.2
Winslow	Petite	136	45.7	45.7	45.7	78.7	96.5	101.6
Winslow	Petite Wide	136	53.0	45.7	45.7	91.0	96.5	101.6
Winslow	Standard	158.7	53.5	48.2	49.5	81.2	101.6	106.6
Winslow	Tall	158	56.0	55.0	55.0	91.0	106.0	115.0

Model	Motors	Control	Lay-flat	Fabric	Colours	Stands out for
Mercer	Quad	USB hand wand	—	Badlands grained leather look	Saddle, Eclipse	The smallest footprint in the range — 81.2 cm wide in Petite. Extending footrest for full lower-leg support.
Studio	Quad	USB hand wand	Yes	Santa Barbara 100% polyester	Fog, Latte, Bark	Inherent stain resistance and the widest Standard seat at 57.1 cm. Independent power headrest and lumbar.
Seagrove	Quad	USB hand wand	Yes	Picasso premium leather look	Walnut, Midnight	Hidden cup holder and a built-in wireless phone charging pad. Rounded arms.
Aroki	Quad	USB hand controller with pre-sets	Yes — 183 cm	Suede look NZ exclusive	Silt	Two-stage lift and a full lay-flat recline to 183 cm. Rated to 158 kg in both sizes.
Winslow	Five	App (iOS & Android) + USB hand wand	Yes	Cappricio / Stonewash	Cappricio Saddle, Cappricio Smoke, Stonewash Dune	Zero gravity, Trendelenburg tilt, seat memory and a power aircraft-style headrest. Four sizes.

Getting the fit right

Seat width — across the hips, seated, plus 2–5 cm.

Seat depth — back of knee to backrest, less 2–3 finger widths.

Seat to floor — feet flat, hips level with or above the knees.

Room — allow clearance behind for the backrest to recline fully.

Choosing between them

Tight on space? Mercer — 81.2 cm wide in Petite.

Spills a concern? Studio, in stain-resistant Santa Barbara.

Sleeping or elevating legs in the chair? Aroki — flat to 183 cm.

Oedema, pressure care, complex positioning? Winslow.

Warranty, delivery and funding

Warranty — frame 5 years · mechanisms 3 years · electronics, handset & cable and fabric 2 years.

Delivery — quoted per order on your address and access. Two-person home delivery with installation and packaging removal available nationwide. Allow 7–10 working days, plus 2–3 rural.

Funding — we quote for Enable, ACC, Accessable and WINZ.

Specifications supplied by Pride Mobility Products NZ, correct at August 2026; Theorem reserve the right to alter specifications without notice. This sheet is a fit-and-function reference, not a clinical recommendation — please confirm suitability with the prescribing occupational therapist.