

## **Derby RAISED TOILET SEAT – 4 inch / 10 cm Standard**

Code: 239-4"

Product weight: 1.3 – 1.5 kg  
Max user weight: 190 kg

### TECHNICAL INFORMATION

W380 x D400 x H100 mm

### FITTING / USER INSTRUCTIONS

1. Undo three knobs under seat and reverse fixing brackets. Turn knobs anti-clockwise until brackets are fully extended.
2. Lift existing seat and place raised toilet seat on to WC.
3. Adjust the brackets to give a tight fit to the top of the toilet. Do not over tighten. Test seat to ensure it is securely fitted.
4. Check tightness of fit from time to time.
5. Clean with non-scratch detergent.

